

SOCKET GRAFT INJECTABLE


- Ready to use
- No mixing required



STORAGE

REFRIGERATE PRODUCT UPON ARRIVAL

Product will be shipped in an insulated shipping box with cold packs. The cold packs may become thawed during shipment. If the gray temperature indicator located on the product packaging is black upon arrival, the efficacy of the graft material may have been compromised. Notify us immediately if the dot is black. Refrigerate product if dot is not black.

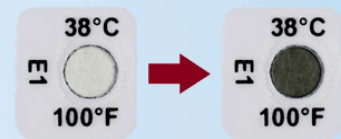

DIRECTIONS


- If significant bleeding occurs before grafting, dry the socket with a sterile 2x2 gauze wetted with 1:50 lidocaine. Wait for bleeding to slow, then quickly fill the socket with Socket Graft Injectable. **Blood does not need to be present.**
- Remove granulation tissue and any foreign material.
- Fill the socket with Socket Graft Injectable beginning at the apex and filling toward the gingival margin. **Do not mix with any other graft materials other than SteinerBio bone grafts.**
- Using a flat instrument, push the graft material down to completely fill the root structure of the socket. **Do not pack the graft material.**
- Cover with a d-PTFE (Teflon) membrane. **Do not use animal or human membranes to cover Socket Graft Injectable or the graft may fail.**
- Oral Bond is advised to be placed over the membrane, creating a hermetic seal to prevent oral fluids and bacteria from entering the grafted site.
- Systemic antibiotics are advised if infection is present.
- If you choose to temporize, use an essex retainer rather than a flipper.

To avoid displacing the graft material, instruct patient to avoid the area with their tongue, and when eating or brushing. No vigorous rinses or spitting.

PRECAUTIONS

Do not overfill. Do not compromise blood supply to the defect area. This product has not been studied in pediatric patients. Socket Graft Injectable is not intended for immediate load bearing.

Federal law restricts this device to sale by or on the order of a dentist.

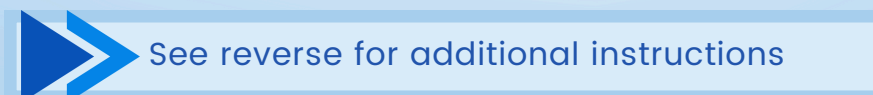

SOCKET GRAFT INJECTABLE

DESCRIPTION

Socket Graft Injectable is a dual phase calcium phosphate based synthetic bone graft material. It is designed to quickly and safely treat the extraction socket, to retain alveolar bone, stimulate bone formation, and reduce post-operative complications. This product is designed to be quickly and completely resorbed, to produce normal, healthy vital bone. This product contains no human or animal tissue.

Ridge preservation insurance code: D7953

INDICATIONS

Socket Graft Injectable is a bone grafting material intended for use in dental extraction sockets that have all walls remaining. This product is cleared by the FDA for implant placement at 8 weeks.

CONTRAINDICATIONS

This product is contraindicated in individuals who are unable to follow post-op instructions.

WARNINGS

Socket Graft Injectable is single use only. Do not re-sterilize or reuse. Do not use Socket Graft Injectable if the package has been opened or damaged, or has exceeded the expiration date. Do not use if the temperature indicator on the packaging has turned black.